REBARON

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DIRECTIONS FOR USE

1. PREPARATION

- a) Check the fit in the mouth using GC FIT CHECKER (II) or a thinly mixed alginate impression material.
- b) Mark the boundary between thick and thin layers using a round bur.
- c) Clean the contact surface thoroughly using pumice and water and abrade the area to be rebased. (Pressure relief holes may be drilled in areas of extensive undercuts).
- d) Protect the tooth and gingival areas of the denture with GC COCOA BUTTER, and apply local anaesthetic or glycerine to the patient's mucous membrane contact areas to prevent irritation.

2. MIXING

- a) The standard powder/liquid ratio is 1.5 g/1.0 cc, (half a measuring cup of powder to 1/3 of a syringe of liquid).

Note:

Excessive quantities of liquid can cause strong irritation.

3. REBASING

- b) Measure into the jar and mix for 20-30 seconds.
- c) 30 seconds after mixing, coat the mixture onto the denture and leave until the mixture has lost its sheen (1 minute 30 seconds).
- b) When the surface is dull, insert the denture into the mouth and make muscle trimming in the centric occlusal position.
- c) Wait at least 2 minutes before carefully removing the denture. Correct the periphery.
- d) Re-insert the denture in the mouth, and retain it in the centric occlusal position for about 2 minutes.

Note:

If the heat generated causes the patient discomfort, allow the denture to cure outside the mouth.

- e) REBARON will cure completely in about 9 minutes from start of mixing. Perform final correction and polishing using standard techniques.

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No. 3 - Pink
No. 4 - Dark pink
No. 8 - Live pink
Pink X

PACKAGES

1. Bottle of 100 g powder.
2. Bottle of 100 g (104 mL) liquid.
3. 1-1 package : 100 g powder, 100 g (104 mL) liquid, plus accessories.

WARNING

1. The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
2. In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 08/2009

CE 0086

,"GC,"

MANUFACTURED by
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED by
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU : GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

GC AMERICA INC.
3737 West 127th Street, Alsip, IL 60803 U.S.A.
TEL: +1-708-597-0900.

GC ASIA DENTAL PTE. LTD.
19 Loyang Way, #06-27 Singapore 508724
TEL: +65 6546 7588

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Prior to use, carefully read
the instructions for use.

EN

REBARON

SELF-CURING ACRYLIC FOR DIRECT REBASING

For use only by a dental professional in the recommended indications.

RECOMMENDED INDICATIONS

For the rebasing (relining) of acrylic dentures.

CONTRAINdications

In rare cases the product may cause sensitivity in some people. If any such reactions are experienced, discontinue the use of the product and refer to a physician.

DIRECTIONS FOR USE

1. PREPARATION

- Check the fit in the mouth using GC FIT CHECKER (II) or a thinly mixed alginate impression material.
- Mark the boundary between thick and thin layers using a round bur.
- Clean the contact surface thoroughly using pumice and water and abrade the area to be rebased. (Pressure relief holes may be drilled in areas of extensive undercuts).
- Protect the tooth and gingival areas of the denture with GC COCOA BUTTER, and apply local anaesthetic or glycerine to the patient's mucous membrane contact areas to prevent irritation.

2. MIXING

- The standard powder/liquid ratio is 1.5 g/1.0 mL, (half a measuring cup of powder to 1/3 of a syringe of liquid).
Note:
Excessive quantities of liquid can cause strong irritation.
- Measure into the jar and mix for 20-30 seconds.

3. REBASING

- 30 seconds after mixing, coat the mixture onto the denture and leave until the mixture has lost its sheen (1 minute 30 seconds).
- When the surface is dull, insert the denture into the mouth and make muscle trimming in the centric occlusal position.
- Wait at least 2 minutes before carefully removing the denture. Correct the periphery.
- Re-insert the denture in the mouth, and retain it in the centric occlusal position for about 2 minutes.
Note:
If the heat generated causes the patient discomfort, allow the denture to cure outside the mouth.
- REBARON will cure completely in about 9 minutes from start of mixing. Perform final correction and polishing using standard techniques.

STORAGE

Store in a cool place away from direct sunlight.
(Shelf life: 3 years from date of manufacture)

SHADES

No. 3 - Pink

No. 8 - Live pink

Pink X

PACKAGES

- Bottle of 100 g powder.
- Bottle of 100 g (104 mL) liquid.
- 1 package : 100 g powder, 100 g (104 mL) liquid, plus accessories.

WARNING

- The liquid is volatile and highly flammable. Close the bottle immediately after use and keep away from sources of ignition.
- In case of contact with eyes, flush immediately with water and seek medical attention.

Last revised : 06/2012

CE 0086

'GC.'

MANUFACTURED BY
GC DENTAL PRODUCTS CORP.
2-285 Torimatsu-cho, Kasugai, Aichi 486-0844, Japan

DISTRIBUTED BY
GC CORPORATION
76-1 Hasunuma-cho, Itabashi-ku, Tokyo 174-8585, Japan

EU: GC EUROPE N.V.
Researchpark Haasrode-Leuven 1240, Interleuvenlaan 33,
B-3001 Leuven, Belgium TEL: +32 16 74 10 00

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